



POCKET CPR MASK

USER MANUAL




Intended Use	2
Product Overview	3
Operation Instruction for Adult & Pediatric	4
Operation Instruction for Infant	5
Technical Sheet	6
Caution	6
Warning	7
Certificate	8
Warranty	8


The FirstCare CPR Mask is designed for mouth-to-mask ventilation of a non-breathing adult, child or infant.

This pocket mask features a one-way filter valve which provide protected rescue breathing for both patients and rescuers. This latex-free PVC resuscitation mask can be used for both adults, children and infants and is stored in a robust and snap-to-open clam-shell plastic hard case with two alcohol prep pads, a pair of gloves and a detailed instruction booklet.


Product Overview

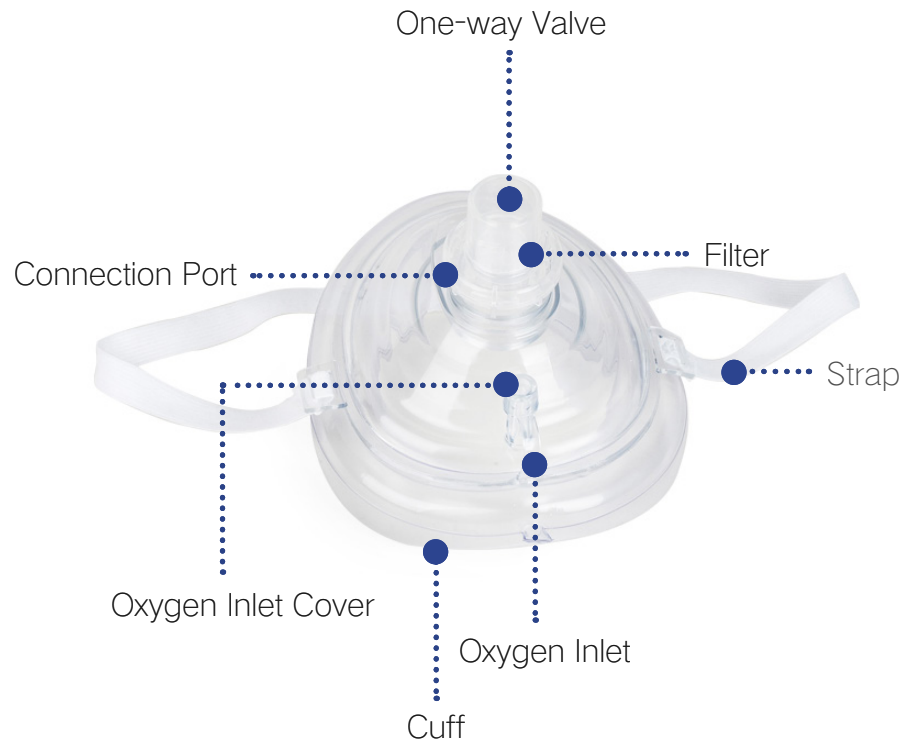


Operation Instruction for Adult & Pediatric

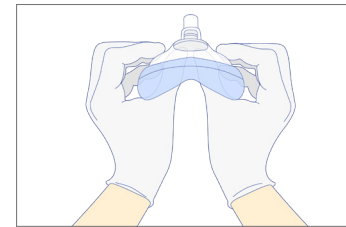


- ① Put on the protective gloves. Unfold mask and check that filter is firmly in place.

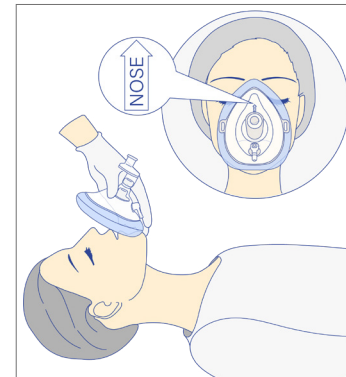


- ② Use the rim of the mask to retract the lower lip so the patient's mouth remains open under the mask. Use thumbs to hold the mask and your fingers to grasp the lower jaw. Position end (nose) over the patient's nose.

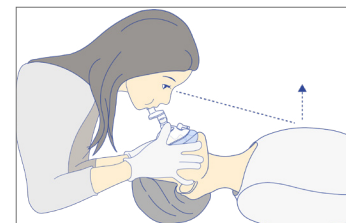


- ③ With face mask sealed as described above, blow slowly into Non-Rebreathing Valve.

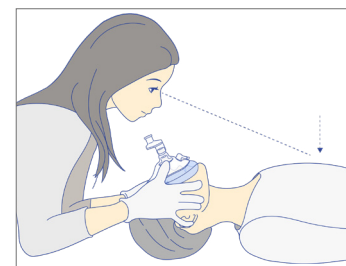


- ④ Allow patient to exhale. Repeat as CPR guidelines.

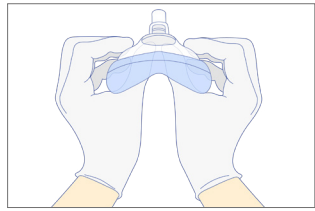


- Put on the protective gloves. Unfold mask and check that filter is firmly in place.

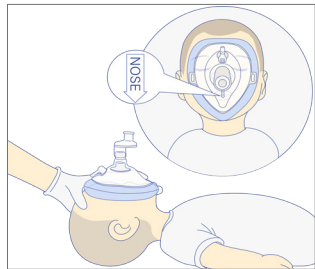

- Use the rim of the mask to retract the lower lip so the patient's mouth remains open under the mask. Use thumbs to hold the mask and your fingers to grasp the lower jaw. Position end (nose) under the patient's chin.

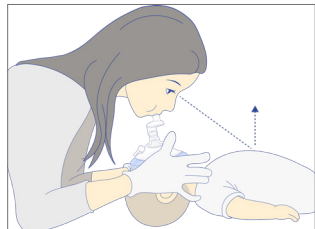

- With face mask sealed as described above, blow slowly into Non-Rebreathing Valve.

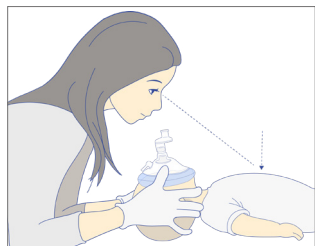

- Allow patient to exhale. Repeat as CPR guidelines.

Oxygen Concentration (%)								
O ₂ Flow (LPM)	0 LPM	2 LPM	4 LPM	6 LPM	8 LPM	10 LPM	12 LPM	14 LPM
O ₂ Concentration %	16.4	34.6	43.2	49.2	53.5	56.7	63.6	67.5

Dimension	123mm x 101mm x 86mm
Inspiratory resistance	0.40KPa-0.49KPa (at 50l/min)
Expiratory resistance	0.20KPa-0.30KPa (at 50l/min)
Operation Temperature	-18°C ~ 40°C
Storage Relative Humidity	< 85%
Storage Temperature	-40°C ~ 60°C

Caution

- Other ventilation equipment, e.g. a bag-valve resuscitator or oxygen powered resuscitator, may be coupled to the 15 mm connection port (without the One-way Valve) of the Pocket Mask.
- At lower temperatures the Pocket Mask can be harder to unfold.
- Use with oxygen: The mask may be used with supplemental oxygen. Attach the oxygen tubing to the optional oxygen inlet and set the flow rate. If the patient is spontaneously breathing, use the head strap to position the Mask on the patient's face. If the patient is NOT breathing, begin mouth to mask ventilation immediately.
- Always keep mask in storage case when not in use.

Warning

- ① The mask should only be used by persons who have received adequate training.
- ② If contaminated, remove mask and clear patient's airway. Clear mask by shaking or sweeping out foreign material. Blow through valve to verify operation. Reapply mask and continue ventilation.
- ③ If the product becomes too contaminated or difficult to clear, continue with ventilation.
- ④ The mask should not be used with supplemental oxygen where smoking is permitted or when fire, flame, oil or grease is in close proximity.
- ⑤ CPR mask should not be used in toxic or hazardous atmospheres.
- ⑥ Use of non-FirstCare parts may affect safety and/or performance.
- ⑦ FirstCare is not responsible for any consequences of re-use of the disposable CPR mask.

Certificate




Warranty

This equipment is manufactured from the finest quality materials. Each individual part is subject to strict quality control tests to ensure exceptionally high standards.

The manufacturer warrants to the purchaser of FirstCare CPR mask that its component parts are free from defects in material and workmanship for a period of one year from the date of purchase. The manufacturer will replace and/or repair all parts of the mask at its option for one year from the date of purchase at no cost to the purchaser, upon the notification of the defects, in writing by the purchaser. All shipping costs shall be borne by the purchaser.

The manufacturer shall be liable under this warranty only if the mask and its parts have been used in the normal manner described in the instruction manual.

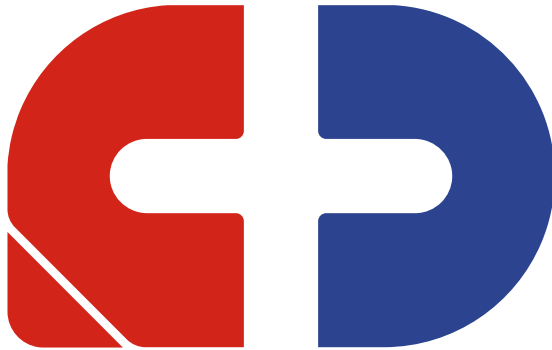

FIRSTCARE™
WE EQUIP REAL-LIFE HEROES

Xiamen FirstCare Imp. & Exp. Co.,Ltd.

📍 1803, BLDG. #1-T3, SEC Taidi Haixi
Center Xiamen, China 361026

🌐 www.firstcaresolutions.com

☎ 0086-592-6531682

✉ info@firstcaresolutions.com

Connect with us:

